

## APEX RADIOLOGY

### INTELEVIEWER INSTALLATION NEW USER

CLICK FOR  
INTELEVIEWER INSTALLERS

CLICK FOR  
APEX CONNECT

CLICK FOR INTELEVIEWER  
QUICK REFERENCE GUIDE

#### How to Install Intelviewer

If you do not have IntelViewer installed please navigate to the following URL:

<https://pacs.apexradiology.com.au/IntelViewer-tracks/stable/>

- Please select the appropriate installation for your Operating System (Windows 32 bit, Windows 64 bit or Mac OS X)
- Once downloaded, an "IntelViewer" shortcut will appear on the computers desktop. Please open IntelViewer via the shortcut and add APEX RADIOLOGY as a bookmark. (Details explained below).

If you already have IntelViewer installed you will need to add a bookmark for Apex Radiology. To do this, please follow these steps:

1. Open IntelViewer
2. On the right hand side, click Edit Bookmarks
3. In the new window labelled "Server Bookmarks" click Add (top, left-hand side)
4. Under "Server URL" type: <https://pacs.apexradiology.com.au>
5. Enter APEX RADIOLOGY as a Custom Server Name to distinguish it from any other bookmarks.
6. Click OK
7. Click Close

You will now be able to log into our IntelViewer servers by selecting the APEX RADIOLOGY bookmark and entering the logon details provided in this email.

If you experience any difficulties, please contact our RIS/PACs team, on: [rispacs@apexrad.com.au](mailto:rispacs@apexrad.com.au) or for urgent requests, phone: 1300 668 957.